



The accuracy of the DrugCheck® SalivaScan™ was evaluated in comparison to commercially available drug screen tests. Thirty (30) negative oral fluid samples collected from presumed non-user volunteers were tested by both DrugCheck SalivaScan and commercially available drug screen tests. Of these negative oral fluid samples tested, all were found negative by both methods.

In a separate study, positive oral fluid samples, obtained from clinical laboratories where the drug concentrations were determined by LC-MS/MS (Liquid Chromatography/Tandem Mass Spectrometry), were tested by DrugCheck SalivaScan and commercial drug screen tests. The overall combined accuracy of the DrugCheck SalivaScan is 96%