



Read instructions before use

“Give Me A Reason” is a drug prevention initiative of Operation UNITE.

Before Starting



- Test kit should be room temperature before use.
- Instruct the donor to **not place anything** in the mouth – including food, drink, gum, or tobacco products – for at least **10 minutes** prior to use.

Collect the Specimen

- Use only the oral fluid collection swab provided.
- Sweep the inside of the mouth (cheeks, gums and tongue) several times.
- Hold the swab inside mouth until color on the saturation indicator strip appears in the indicator window of the collection swab.
- DO NOT bite, suck, or chew on the sponge.
- Refrain from talking while collection swab is in the mouth.
- After 7 minutes, proceed with the test below, even if color on the saturation indicator strip has not appeared in indicator window.

Proceed to Test

- Place the test container on a flat, hard surface.
- Remove the collection swab from the mouth and insert it, sponge first, into the screening device.
- Press cap down tightly until fully locked. The device is now airtight and tamper evident.
- Keep the test kit upright.
- Wait for the colored bands to appear in the test results area.
- Read results at 10 minutes (3 minutes for “ALC”).
- Do not interpret the results after 20 minutes.

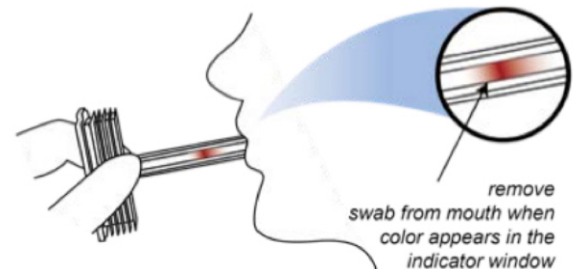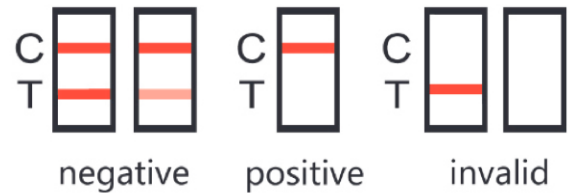


What the Panels Indicate



COC	Cocaine
OPI	Opiates
THC	Marijuana
METH	Methamphetamine
AMP	Amphetamine
PCP	Phencyclidine
BENZO	Benzodiazepine
BARB	Barbiturate
K2	Spice (synthetic cannabinoids)
FEN	Fentanyl
BUP	Buprenorphine
OXY	Oxycodone
METHA	Methadone
ALCO	Alcohol 0.02%

Read ALC Results at 2 Minutes

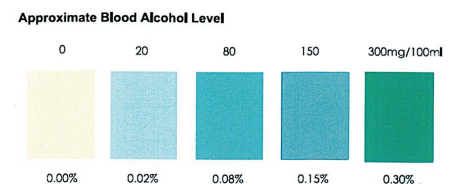

Please help us evaluate the effectiveness of this initiative by returning the enclosed card to **350 C.A.P. Drive, London, KY 40744**, or by completing the survey on-line.

OperationUNITE.org

All information is completely anonymous.